

Rhino Crush™ Tablet Crusher

A Powerful Crush with Minimal Effort - Removes OH&S Worries



A powerful crush with a gentle touch, the Rhino Crush uses the same amount of force as 90 Rhinos standing on top of each other.



Crushing medications can now be a quick, effortless task with the help of Webstercare's NEW Rhino Crush. The unique crushing mechanism and crush head ensures the effort required is minimal. The crushing action ensures medications are crushed into a fine powder to eliminate swallowing difficulties or blocked feeding tubes. As well as minimal effort there is minimal noise produced meaning less disturbance to residents. The Rhino Crush has been ergonomically reviewed to reduce risks

of OH&S injuries. It can be easily cleaned, is durable and lightweight. The Rhino Crush can be attached to any trolley with the use of a Rhino Crush Clip.

For best crushing results, Webstercare recommends the use of the Webstercare Crush Cups. Placing the medication between two crush cups will prevent cross contamination and reduce medication dust particles being released in the air.

Rhino Crush™ Tablet Crusher



Instructions for use



STEP 1

Place medications between two crush cups. This will prevent cross contamination and ensure no medication powder is lost.



STEP 2

With minimal effort, press the handle downward to crush the tablets. For best results crush 3 or 4 times, rotating the cups as you crush. NOTE: Do not use force to push down on the tablets. Rather, use minimal effort and rotate cups until a fine powder is achieved.



STEP 3

Once the medications are crushed, place suitable vehicle (eg. Puree) into the crush cup ready for administration.

- Minimal effort required
- Reduce OH&S risks
- Minimal noise
- Crushes to a fine powder
- Easy to clean

ORDER NOW – Product Code: 701

TO ORDER: Phone 1800 244 358, Fax 1800 626 739 or email orders@webstercare.com.au

FOR MORE INFORMATION ON RHINO CRUSH PLEASE CONTACT WEBSTERCARE ON

➤ webstercare.com.au

1800 244 358 | info@webstercare.com.au | 17-19 Moore St, Leichhardt NSW 2040

 **Webstercare®**